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Attention: Please complete and sign this form as proof of disinfection and add it to your shipment. If proof of disinfection is missing, you will be charged a cleaning and disinfection fee of 45.00 EUR!

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Cleaning

Wipe the colposcope, the stand or the article with a damp (not wet) cloth. Standard solutions in the usual thinning can be used for cleaning. Then wipe with a dry, lint-free cloth. To avoid damage to the equipment, observe the following instructions for cleaning:

- For cleaning of the lenses you can use an aqueous soap solution. For drying of these you should use special optical cloths.
- Never use strong or abrasive cleaning agents! This results in damage to the surface.

Disinfection

Disinfect the unit with conventional disinfectants damp (not wet) in accordance with national hygiene standards and in accordance with the internal hygiene plan. The cleaning solutions and disinfectants have to be used in accordance with the requirements of their manufacturer and the required soaking time.

Attention! The power supply is not protected against the penetration of liquids. Penetrating liquid may cause electrical shock.

- Avoid drop and splashing water, as this may cause damage to the device.
- Do not spray the power supply with cleaning or disinfecting agents.

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